

Your Advantages:

- Reduces spread of droplets
- Comfortable
- Desinfectable
- Reusable
- Light and robust
- Cost effective



Please Contact

Kayser Filtertech GmbH
Baustraße 38
37574 Einbeck

www.protectionmask4u.com
masks@kayser-filtertech.com

Tel.: +49 (0) 5561 79020

Mouth-Nose-Cover with Replacable Inlay



Product description

- Robust plastic face piece with elastic bands
- Ergonomic design
- Easily replacable and reusable inlay made of:
 - 70% Polypropylen / 30 % Polyester (food conformity according to EU-VO 10/2011, 1935/2004 and FDA 21 CFR §177.1630)
- Cleanable and desinfectable
- Machine washable at 60°C (use laundry net, do not iron)

Usage:

- Thoroughly wash and disinfect hands prior to donning and removing.
- Do not touch the cover while wearing.
- Wear over mouth and nose.
- In case of moisture penetration, remove and air-tightly pack, replace inlay if necessary.
- Wash used inlays as soon as possible. Do not iron.
- Observe safe distance of 1.50 m while wearing

Not a medical product according to DIN EN 14683


Not personal protection equipment according to DIN EN 149:2001-10,

Does not protect against infection by the SARS-CoV-2 virus or other viruses or bacteria


ATTENTION: Keep out of hands of children under 6 years of age

Assembly instructions overleaf


Make sure that the plastic face piece is clean and disinfected. Disinfect your hands so that the filter material is not contaminated.

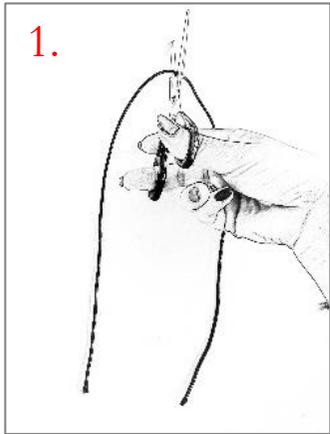


1. Cut the elastic band in the middle.

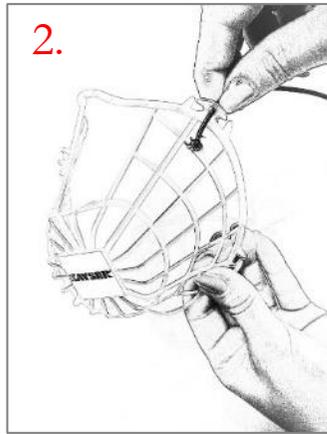


2. Pull the band through the upper hole of the plastic face piece and knot it in front of the hole.

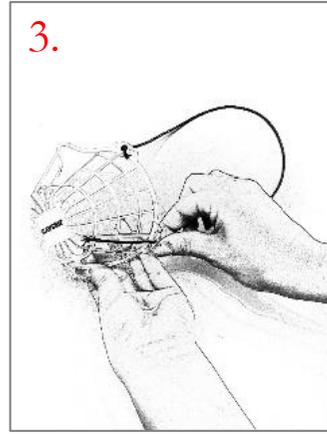


3. Now thread the band through the hole below.

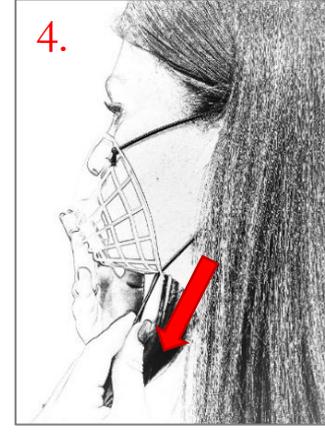


4. Put the mask on to determine the length of the tape. The mask should not be too tight.

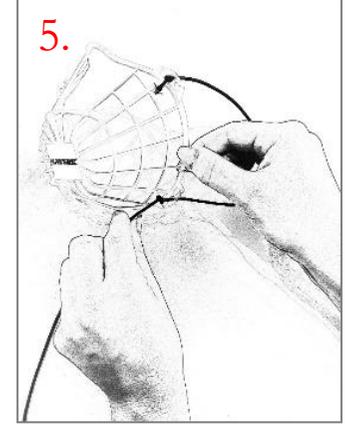


5. Also knot the band in front of the hole. In order to shorten the band afterwards, it is possible to wrap the band around the holding brackets.

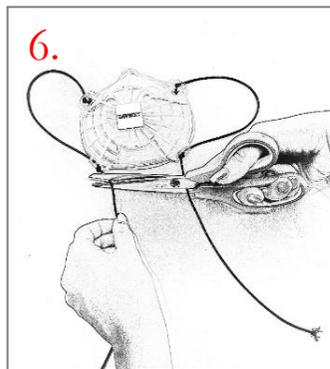


6. Repeat the procedure with the second band on the other side. Cut off the remaining band.

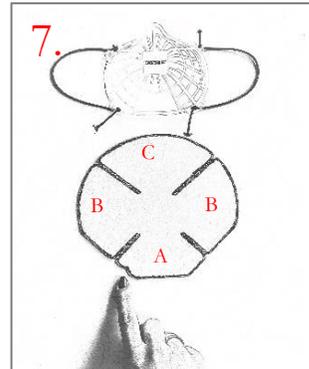


7. Place the inlay with the straight cutting edge in front of you. Make sure that the notch is on the left side.

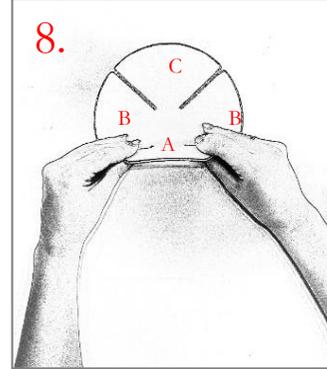

8. Press the inlay into the plastic carrier so that the side tabs (B) overlap the top tab (C) and the bottom tab (A) is above the side tabs (B). The plastic face piece must be completely covered by the inlay.

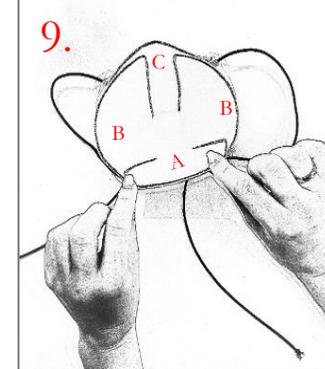

9. Fold the side with the straight cut edge upwards. Take the mask.

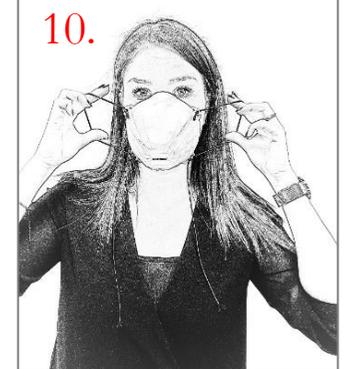

10. Now put on the mask. Provided that you do not feel any pressure points and the filter support is close to your face everywhere, it will fit correctly.